

Quick & easy, self-collected COVID-19 testing to fit your needs.

1

FDA AUTHORIZED EUA*

Emergency Use
Authorization (EUA)
for self-collected RT-
PCR SARS-CoV-2
test kit.

2

HIGHLY SPECIFIC

During validation, no
false positives and no
cross-reactivity from
viruses that cause the
common cold.

3

HIGHLY SENSITIVE

High sensitivity
revealed during the
initial laboratory
validation study.

4

DETECTS VIRUS

A positive test
result means that
the COVID-19 virus
was detected in
the nose sample.
Detects as low as
180 NDU/mL.



5

48 HR RESULTS

Orders are shipped
overnight & test
results are ready
48 hours after
return shipping on
our secure, online
portal.

6

PAINLESS SWAB

The shallow & pain-free
self-collected nasal
swab you've been
waiting for. No more
deep discomfort.

7

RT-PCR TECH

Reverse transcriptase
polymerase chain
reaction is used to look
for SARS-CoV-2 RNA in
the nose.

8

RETURN SHIPPING

The empowerDX kit
includes a pre-labeled
return shipping
envelope for your
convenience.

*Not available in NY, NJ, RI & MD. Individuals being tested must be 18+ years old. This home-collection kit has not been FDA cleared or approved; rather it has been authorized by FDA under an EUA only for the self-collection and maintenance of nasal swab specimens as an aid in detection of nucleic acid from SARS-CoV-2, and not for any other viruses or pathogens, and only for the duration of the declaration that circumstances exist justifying the authorization of emergency use of in medical devices during the COVID-19 outbreak under Section 64(b)(1) of the Federal Food, Drug and Cosmetic Act, 21 U.S.C. § 360bbb-3(b)(1), unless the authorization is terminated or revoked sooner.

COVID-19 TESTING OPTIONS, TAILORED TO YOU

1

SWAB PARTY

*Bulk delivery + group collection
+ bulk return*

Ideal solution for...

All customers needing to test a large group at one time.

Collection

Self-collected at the dedicated site on a specific date.

Shipping

Kits are bulk shipped /returned. Overnight M-F.
No Sat returns.

Results

Individual gets an email notification to access results. Dashboard analytics for customer.

2

GRAB & GO

Bulk delivery + individual collection + individual return

Ideal solution for...

Corporations that store kits for ad-hoc use. Back to business travel.

Collection

Self-collect anywhere. Online registration & consent.

Shipping

Bulk shipped to customer. Individuals FedEx return M-F (not Sat).

Results

Individual accesses results online at my.empowerdxlab.com. No customer dashboard analytics.

3

INDIVIDUAL

Individual delivery + individual collection + individual return

Ideal solution for...

Cruise ship customers that require passenger screening.

Collection

Self-collect anywhere. Online registration & consent.

Shipping

Kit shipped to individual. FedEx return M-F (not Sat).

Results

Individual accesses results online at my.empowerdxlab.com. No customer dashboard analytics.




We'd love to hear from you: partner@empowerdxlab.com.

WHAT'S INSIDE YOUR KIT


Collection Instructions

8-1/2" x 11"

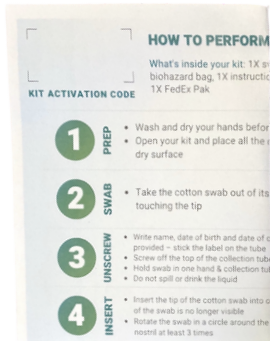

The Kit

6-3/4" x 5-11/16" x 1-13/16"

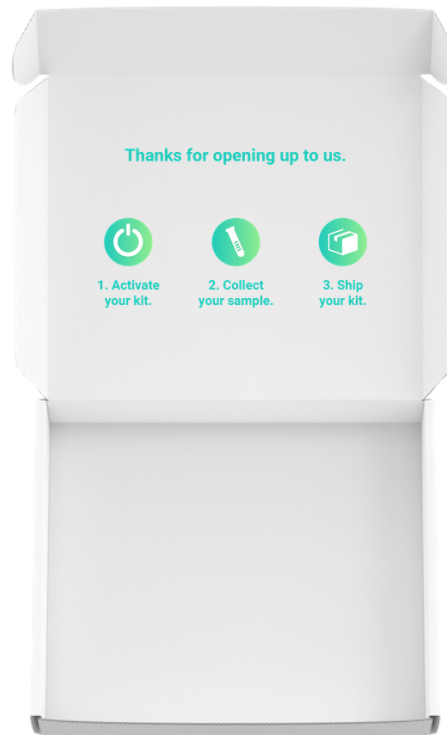


FedEx Return Pak

UN 3373 Labeled



Nose Swab

Fisher Scientific
3" Sterile Polyester
Spun Swab

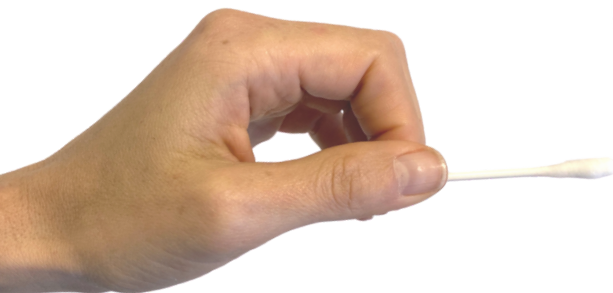


Saline Tube (Plastic Container)

Azer Scientific

10 mL size with 3mL of Sterile
Phosphate Buffered Saline

Biohazard Bag

To Store the Sample

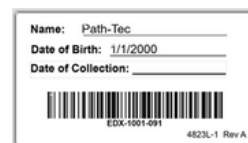

Name Label

Unique Barcode

HOW TO PERFORM YOUR NASAL SWAB

What's inside your kit: 1X swab, 1X plastic container, 1X biohazard bag, 1X instructions (keep this sheet), 1X name label, 1X FedEx Pak

COVID-19
TEST KIT

empowerDX

KIT ACTIVATION CODE
activate kit before you ship

1

PREP


- Wash and dry your hands before opening the kit box
- Open your kit and place all the contents on a clean, dry surface
- Failure to follow these instructions may lead to inaccurate test results – please keep this sheet for your reference



2

SWAB


- Take the cotton swab out of its package without touching the tip

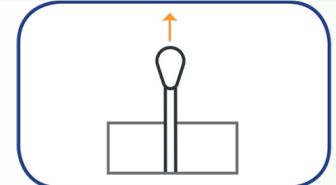

3

UNSCREW


- Write name, date of birth and date of collection on the label provided – stick the label on the tube
- Screw off the top of the collection tube
- Hold swab in one hand & collection tube in the other
- Do not spill or drink the liquid

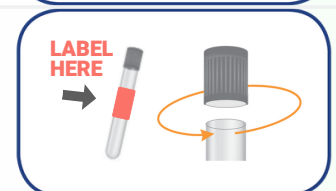

4

INSERT


- Insert the tip of the cotton swab into one nostril until the cotton tip of the swab is no longer visible
- Rotate the swab in a circle around the entire inside edge of your nostril at least 3 times

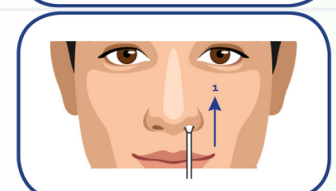

5

REPEAT


- Take the cotton swab out of your nostril
- Using the same end of the cotton swab, repeat step 4 in your other nostril

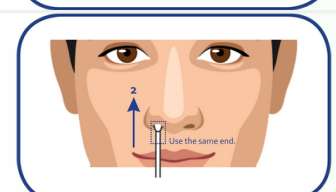

6

SCREW

- Remove the cotton swab from your second nostril
- Place the used end into the collection tube first so it touches the liquid
- Screw on the top of the collection tube

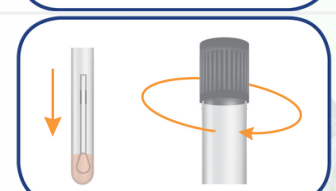

7

INSERT


- Wash and dry your hands thoroughly again
- Insert collection tube into the biohazard bag, leave the white absorbent pad in the bag and close the zip lock seal
- Place the biohazard bag into the kit box



8

ACTIVATE

- Activate your kit **today** by clicking the link in the empowerDX email you received and entering the 10 digit kit activation code (or) go directly to activate.empowerdxcab.com in your browser
- **Your kit MUST be activated to receive your test results**

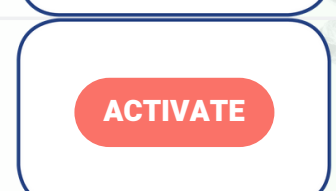

9

SHIP

- Place the kit box in the pre-labeled FedEx Pak, remove the adhesive cover strip and seal the FedEx Pak closed
- Ship the FedEx Pak by (1) ordering a FedEx pickup or (2) finding the nearest FedEx drop box: bit.ly/fedexdropbox (Monday-Friday only)
- Ship on the same day as collection or your sample may not be tested





LABORATORY REPORT

Patient Name: SAMPLE, REPORT
Sex: M **DOB:** 05/27/1937 **Age:** 83y
Viracor Eurofins Patient ID: 8443217
Physician: Unknown, Dr.
Received: 09/28/2020 09:19
Report Delivered: 09/28/2020 10:37

Client MRN: 8046359

Client:
1234 1st Street
Suite B
Anywhere, Mo 11223

RESULTS

At Home COVID-19 PCR (Nasal Swab)

Viracor Eurofins Accession ID: 2027200982

Performed at: Viracor Eurofins - 1001 NW Technology Dr., Lee's Summit, MO | CLIA# 26D-0983643

Client Accession ID:

Collected: 09/28/2020 08:20

Final - Approved 09/28/2020 10:34

TEST	RESULT	FLAG	REF RANGE
SARS-CoV-2 RT-PCR	Positive	A	Negative



LABORATORY REPORT

Patient Name: SAMPLE, REPORT
Sex: M **DOB:** 05/27/1937 **Age:** 83y
Viracor Eurofins Patient ID: 8443217
Physician: Unknown, Dr.
Received: 09/28/2020 09:19
Report Delivered: 09/28/2020 10:37

Client MRN: 8046359

Client:
1234 1st Street
Suite B
Anywhere, Mo 11223

RESULTS

At Home COVID-19 PCR (Nasal Swab)

Viracor Eurofins Accession ID: 2027200982

Performed at: Viracor Eurofins - 1001 NW Technology Dr., Lee's Summit, MO | CLIA# 26D-0983643

Client Accession ID:

Collected: 09/28/2020 08:20

Final - Approved 09/28/2020 10:34

TEST	RESULT	FLAG	REF RANGE
SARS-CoV-2 RT-PCR	Negative		Negative