



Braces for Patients | Created by Doctors

Offer customized care to your patients with products that were developed by a board certified orthopaedic surgeon.

Doctor In The House braces and medical devices are doctor developed and patient tested. Dr Scott Fried and his team of healthcare professionals are dedicated to creating the best braces around you for healing orthopaedic injury.

These are dynamic, semi-flexible, stepdown braces which are easily fitted, adjusted and molded to your patients in minutes providing superior care and the only true customizable professional grade complete brace line available today.

Our Customizable Step Down Braces

Designed by an Orthopaedic Surgeon to Heal your Patients!

Back Bandit Series L0456 and L0457



Size: Moldable and customizable for optimal comfort and support

The Back Bandit Pro is designed with comfort in mind. It allows protected motion with its adjustable posterior, lateral and anterior frame/panels to aid in healing while living life with controlled motion, decreased pain, and with optimal comfort.

Miracle Back Series L0650 and L0637



Size:
Small 20" - 30"
Medium 30" - 40"
Large 40" - 50"
X-Large 50" - 60"
Universal Belt 20" - 60"

The Miracle Back is a unique LSO brace to help patients heal from back injuries, fractures and sprains without surgery and to help rehabilitate post operative back patients to optimal health and lifestyle.

Back Bandit Excel and Ultima L0458 and L0459 | L0464 and L0465



Size: Moldable and customizable for optimal comfort and support

The Back Bandit Excel and Ultima are unique TLO braces developed by an orthopaedic surgeon to help patients heal from back injuries, often without surgery! The Ultima allows more rigid 4 panel-tripolar, fully customizable support, decreasing acute pain with optimal comfort. Both help rehabilitate fracture and post operative patients with our user friendly, patented, fully adjustable, step-down and heat moldable panels.

Neck Free Shoulder Arm/Abduction System L3960



Size: One size fits all (right or left)

Recovering from shoulder injury or surgery has never been more comfortable and safe. This is the first complete Shoulder System designed to prevent frozen shoulder. Provides maximum support and sequential freedom in treatment for patients with Neck, Shoulder, & Arm Injury.

The Miracle Splint Pro and Pro Dynamic L3915 AND L3916



Size: Standard fits S,M,L glove sizes XL fits XXL hands

The Miracle Splint Pro Series is the answer to every hand/wrist and arm problem. It helps heal Carpal Tunnel, Tenosynovitis, Wrist injury, Arthritis, and Tennis Elbow. Its unique patented heat moldable night rest piece, removable panels and locking hinge are game changers!

The Doctor's Ankle Brace L1906 and L1971



Size: Standard fits S,M,L

How to Apply:
Unfasten top closure cuff and soft liner. WHILE SEATED, place the injured foot in the brace, similar to putting on a shoe. Position so that the heel is against the back of the soft liner.

Fasten the velcro closure on the soft liner. Position so that the rigid plastic stays on the brace are centered over side of the ankle.

Fasten the cuff closure straps over the soft liner so that the brace fits snug and little movement is experienced. Simultaneously pull

both speed-lace straps tight and secure by crossing each speed-lace strap over the top of the foot and attaching to the soft liner. Make sure the brace still fits snug and is centered over each side of the ankle.

The Miracle Knee L1832 and L1833



Size:
Universal size - fits a wide variety of patients

How to Apply:
Unfasten the front closure straps. Place the brace around the knee so that hinges line up with the center of the kneecap on both sides of the leg, and secure the top closure.

Secure the bottom closure using the velcro tab. Keep the hinges aligned with the center of the knee and maintain a tight, snug fit.

Adjust posterior closures if needed for further customized fit and comfort.

Wrap on and apply suspension sleeve for extra support and to avoid brace slipping (see instructions included with the sleeve)

The Doctor's OA Knee Brace L1852




Size:
Universal size - can be customized to the patient

How to Apply:
Open all straps and sit with the leg in a slight bend. Center the hinge about one (1) inch above the top of the knee cap on the inside, just a little toward the back of the knee.

Attach the first strap label on the bottom then the one on the top. Follow with the rest of the straps. Pull the strap snug on top of the knee area. Next, adjust the rest of the straps, making sure they do not feel too tight but securing them snugly so that the brace does not slide.

Stand up and walk, adjust as needed and test bending the knee and walking. For more security add the knee sleeve. Once set just use the easy clips to take off and on.

RespiMask A7032, A7033, A7034, A7035

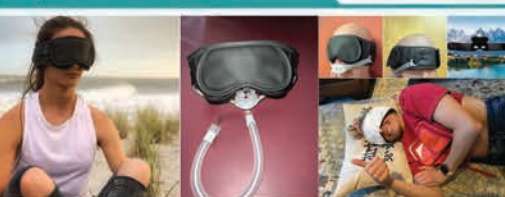

A Sleep Apnea Game Changer!

Improve CPAP compliance with relaxation, better sleep, and less anxiety

Size: One size fits all Standard and XL

- Snoring/Sleep Apnea
- Neck Pain/Nerve Pain
- Improved Respiration
- Sleep Disorders
- Noise Reduction
- Stress/Anxiety

PHONE
610-277-1990 x 151

ADDRESS
721 Skippack Pike, Building B, Suite 103
Blue Bell, PA 19422
United States

ONLINE
askdoctorscott1@gmail.com
docinthehouse.com

